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Ziegelei 1
D-72336 Balingen
email: info@kern-sohn.com

Phone: +49-[0]7433- 9933-0
Fax: +49-[0]7433-9933-149
Internet: www.kern-sohn.com

Operating instructions

Mechanical height measuring rod

KERN MSB 80

Version 1.0
10/2009
GB

MSB_80-BA-e-0910

KERN MSB 80

Version 1.0 10/2009

Operating instructions

Mechanical height measuring rod

Table of Contents

| | | |
|---|--|----|
| 1 | Technical Data | 3 |
| 2 | Declaration of conformity | 4 |
| 3 | Explanation of the graphic symbols | 6 |
| 4 | Basic Information (General)..... | 7 |
| 5 | Basic Safety Precautions | 8 |
| 6 | Unpacking/installing..... | 9 |
| 7 | Body height measurement..... | 10 |
| 8 | Cleaning, Maintenance, Disposal | 11 |

English

1 Technical Data

| KERN | MSB 80 |
|---|---------------------------------------|
| Measuring range | 10 – 80 cm |
| Readability | 1 mm |
| tolerance | 5 mm |
| Dimensions (w x h x d) mm | 890 x 100 x 330 mm |
| Temperature range | +10°C ... +40°C |
| Storage temperature | - 20°C ... + 60°C |
| Net weight | approx. 700 g |
| Admitted as medical product as per Guideline 93/42/EWG (CE 0297) | Category I with measuring function |

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Declaration for appliances with CE mark

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GB We hereby declare and assume sole responsibility for the declaration that the product complies with the standards hereinafter.

The product was manufactured subject to all norms and complies with the standards stated. The products are in conformance with the basic requirements specified in Appendix I of EC Directive 93/42/EC and we are therefore entitled to apply the CE 0297 mark and to put these products into circulation. The appraisal procedure was carried out as per Directive 93/42/EC, Appendix V. The QA system is supervised by the notified body DQS GmbH, Frankfurt/Main, Germany, abbreviation 0297.

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3 Explanation of the graphic symbols

SN WY 090563

Designation of the serial number of every device, applied at the device and on the packaging.

Number here as example

2009-08

Identification of the manufacturing date of the medical product.

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
Identification of manufacturer of medical product including address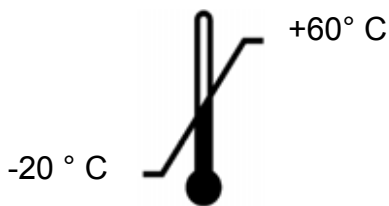

Temperature limitation indicating the upper and the lower limit (storage temperature on packaging)

Temperature indication as example

50.00°F /104.00°F

Temperature limitation in the application indicating the lower and the upper limit (Allowable ambient temperature)

Temperature indication as example

4 Basic Information (General)

This height measuring rod according to guideline 93/42/EWG is provided for determination the body height in medical practice and for medical survey, examination and treatment.

Defined purpose

Indication:

- Determining the body height in the medical practice area.
- At an upright standing person, or at a lying baby, the height measuring rod is applied to head and feet in a manner that on the display or scale the body height value can be read.

Contraindication:

No contraindication known.

Proper use

This height measuring rod is used to determine the height of persons while standing upright, or of lying babies, according to its model, in medical treatment localities. This height measuring rod is suitable to detect, prevent and survey diseases.

Ensure that the height measuring rod may only come into contact only with uninjured skin.

Improper Use

The structure of the height measuring rod may not be modified. This may result in incorrect measuring results, safety-related defects as well as destruction.

The height measuring rod may only be used according to the described conditions. Other areas of use must be released by KERN in writing.

Warranty

Warranty claims shall be voided in case

- Our conditions in the operation manual are ignored
- The appliance is used outside the described uses
- The appliance is modified or opened
- Mechanical damage or damage by media, liquids, natural wear and tear
- Non correct implantation or assembly.

Test device control

For height measuring rods, we recommend a metrological examination of the accuracy of the measuring rod, however, this is not mandatory as the determination of human body height involves rather large, intrinsic inaccuracies.

5 Basic Safety Precautions

Carefully read this operation manual before setup and commissioning, even if you are already familiar with KERN products.

Staff training

The medical staff must apply and follow the operating instructions for proper use and care of the product.

Avoiding contamination

In order to avoid crossed contamination (fungal disease,...) the parts coming into contact with the patient must be cleared regularly.

Recommendation:

After every application which could have a potential contamination as consequence (e.g. direct contact with the skin).

Safety Instructions

- Observe operating instructions.
- Please use no other than the supplied fixtures for mounting.
- During assembly ensure that the height measuring rod is correctly assembled, (see chap. 6).
- When moving the height measuring rod ensure that the person to be measured can not be injured
- The body height measuring emits only reliable values, when the back and the head are straightly aligned, nevertheless the body height of a person may change about some cm during a day.

6 Unpacking/installing


Testing upon acceptance

Please check packaging as well as the device on receipt for possibly visible external damages.

Packaging/Return


Keep all parts of the original packaging for a possibly required return.
Only use original packaging for returning.

Installation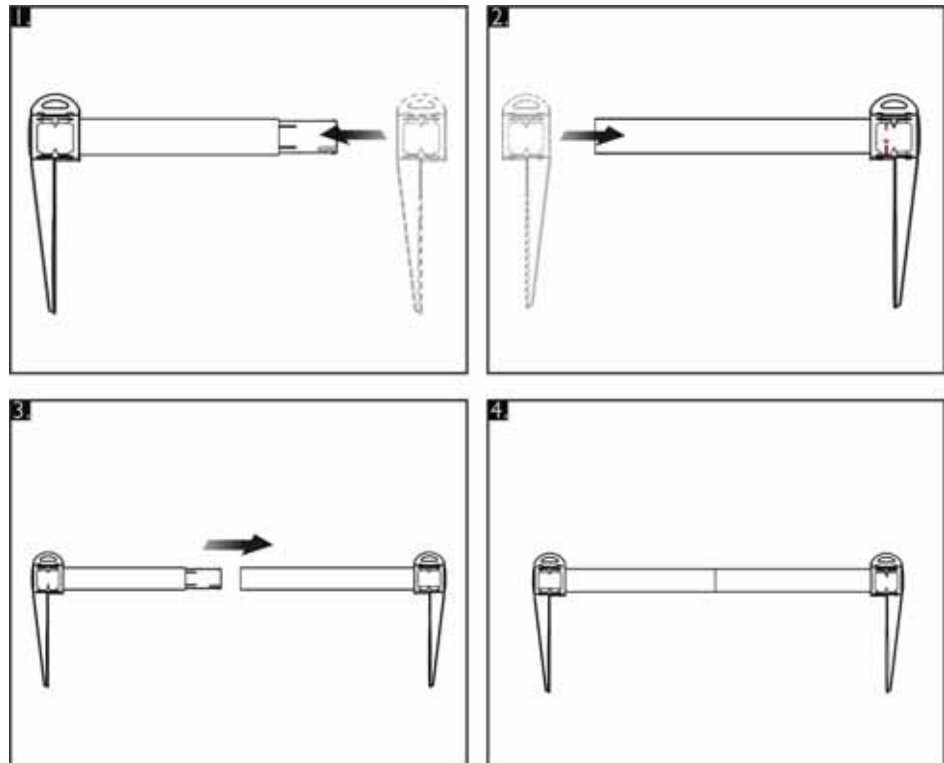


7 Body height measurement

Left-hand use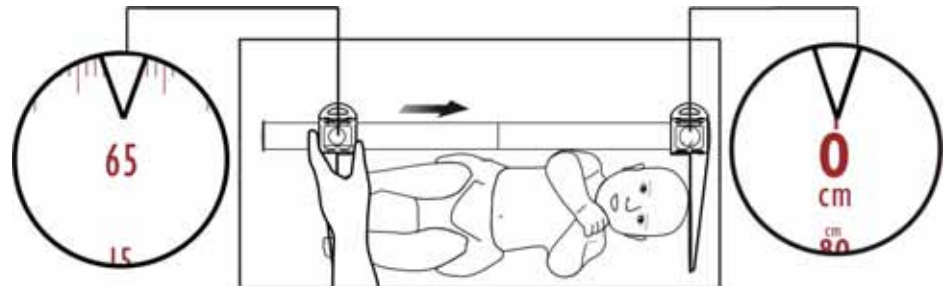

- ⇒ Shift right caliper gauge to the end of the measuring rod to 0 cm.
- ⇒ Shift the left caliper gauge until the stretched feet of the baby are reached.
- ⇒ Read the height on the marking of the left caliper gauge.

Right-hand use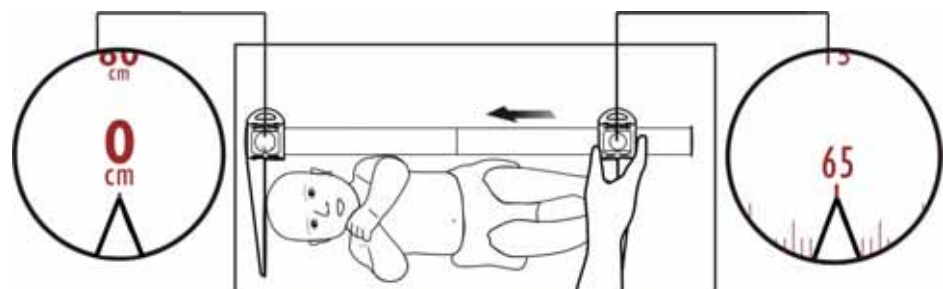

- ⇒ Shift left caliper gauge to the end of the measuring rod to 0 cm.
- ⇒ Shift the right caliper gauge until the stretched feet of the baby are reached.
- ⇒ Read the height on the marking of the right caliper gauge.

8 Cleaning, Maintenance, Disposal

| | |
|-----------------------------|--|
| Cleaning | Please do not use aggressive cleaning agents (solvents or similar agents), but a cloth dampened with mild soap suds, or household cleaner. |
| Disinfection | For disinfection the following substances are allowed: <ul style="list-style-type: none">▪ Methylated spirit▪ 2 % Kohrsolin▪ 1% Sokrena solution▪ 5% Sagrotan▪ 5% Gigasept |
| Service, maintenance | The appliance may only be opened by trained service technicians who are authorized by KERN. |
| Disposal | Disposal of packaging and appliance must be carried out by operator according to valid national or regional law of the location where the appliance is used. |